


## Installation Instructions for Assist Shower Seats

Models	JSS001	Shower Seat 960
	JSS003	Shower Seat 600

NOTE: Both these seats can be used in Assisted Living situations

The Assist Shower Seat 960 can be installed to meet the requirements of AS1428.1 - Design for access and mobility - General requirements for access

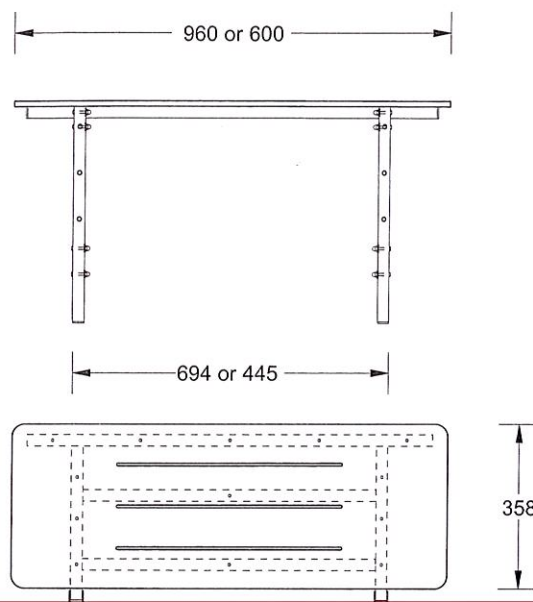

### Installation

1. Open the folding shower seat and locate in position
2. Ensure that the seat is level and that the legs are fully extended to the floor
3. Mark and drill the six holes for the wall fixings
4. Screw the fixings to the wall ensuring a secure fit
5. Check the opening and closing operation

